

Call (888) 401-9998 Option 1 for assistance 24hrs / 7 days a week

Step 1

Turn on mobile device

Press black button on right side to turn on the mobile device



Step 2

Activate Samsung Kiosk App

Open the Host. This will allow CorVitals to remotely access the device.



Step 3

Activate CorVitals ECG App

Click the cloud on the lower right of the of the application. Whenever possible, please connect to Wi-Fi Network. If no network is available, the device will transmit data via the cellular service already installed. Click to accept the terms of monitoring. Enter in the number of days of the study. Click OK, then click register.




Step 4

Activate CVStrip

Before opening package, hold down the power button for 10 seconds, then let go.

A green LED will flash once.

Open package

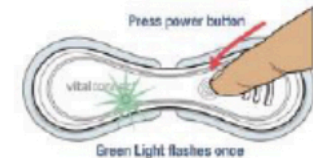

Step 5

Rectangular Snip

Prepare skin area

Use the included alcohol swabs to prepare your skin for the patch application. Allow skin to air dry.


Use the included Cavilon No Sting Barrier Film to prepare your skin for the patch application. Allow skin to air dry.



Step 6

Place patch on chest

As shown below, place the patch on your chest at a 45 degree angle and press firmly along the edges to ensure proper adhesion. You may use adhesive or surgical tape around the edges of the patch only. Do not cover the power button or the three vents above it.

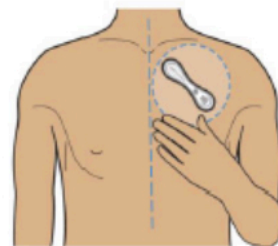

Step 7

Activate connection

Press module icon in the bottom left of the screen and click to scan the patch serial code. This will sync the patch with the mobile device.

Then icon will turn green in 45 seconds. Note that the 99% battery level (not N/A).



Step 8

Ready to go

Verify that Heart Module in the top center, the module icon in bottom left, and cloud icon in the bottom right are all green and the patch is properly applied.

The Headset icon on the right is Technical Support, and the Data Entry icon to the left is to enter any symptoms you may experience during the duration of the monitor.

