

Call (888) 401-9998 Option 1 for assistance 24hrs / 7 days a week

Step 1

Remove CVStrip

When the battery level below the Module Icon in the lower left reaches 3%, it is time to replace the patch.

Slowly remove the strip from the patient. Be sure to not injure the skin while removing.



Rectangular Snip

Step 2

Dispose of CVStrip

Cut the CVStrip down the middle with scissors and dispose of it. This is a recyclable item



Step 3

Open Corvitals App

Open the CorVitals App on the mobile device.




Step 4

Active the CVStrip

Before opening package, hold down the power button for 10 seconds, then let go.


A green LED will flash once. Open package

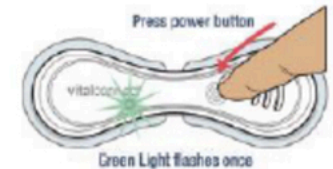

Step 5

Enter the Module Screen

Click on the Module Icon in the bottom left of the screen.

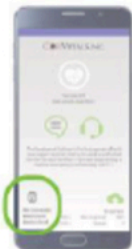

Step 6

Sync patch with mobile device

Sync module and app by selecting the serial number of the new patch.



Step 7

Prepare skin area

Use the included alcohol swabs to prepare your skin for the patch application. Allow skin to air dry.

Use the included Cavilon No Sting Barrier Film to prepare your skin for the patch application. Allow skin to air dry.



Step 8

Place patch on chest

As shown below, place the patch on your chest at a 45 degree angle and press firmly along the edges to ensure proper adhesion. You may use adhesive or surgical tape around the edges of the patch only. Do not cover the power button or the three vents above it.

